

PULPDENT®

Sparkle™

Diamond Polishing Paste

 4 x 1.2mL Sparkle Diamond Polish

R_x only



PULPDENT Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A

www.pulpdent.com • pulpdent@pulpdent.com

PULPDENT®

Sparkle™ Diamond Polishing Paste

**For polishing
porcelain, gold,
metal and
composite to
a high luster
finish**

Sparkle is specially formulated to produce unsurpassed results on porcelain, gold, metal and composite. It does not splatter, and it washes off easily.

Sparkle is packaged in syringes for easy dispensing. It is available in kits of four 1.2ml syringes, and it is also available in a 3ml syringe.

**DIRECTIONS
FOR USE**

1. Apply to surface or restoration with brush, rubber cup or felt wheel.
2. Polish to high luster finish.
3. Wash off with water.
4. Keep syringe capped when not in use.

PULPDENT Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A.

www.pulpdent.com • pulpdent@pulpdent.com

XK-SPARK-IN-07
REV: 03/2018