PULPDENT Corporation • 80 Oakland Street • Watertown, MA 02472 • U.S.A.
 www.pulpdent.com • pulpdent@pulpdent.com

PULPDENT®
Seal Rite Pit and Fissure Sealant

Fluoride Releasing • Light Cure • Radiopaque • Available in Two Viscosities • Off-white shade

PRODUCT DESCRIPTION
 Seal-Rite Pit & Fissure Sealant is a resin-based sealant that releases fluoride and contains no Bis-GMA, no Bisphenol A, and no BPA derivatives. Available in two viscosities, Seal-Rite is 34.4% filled and Seal-Rite Low Viscosity is 7.7% filled. Both are radiopaque and are off-white shades.

INDICATIONS
 For sealing pits and fissures of teeth

CONTRAINDICATIONS
 Not for direct placement on the exposed pulp

INSTRUCTIONS FOR USE

1. Thoroughly clean enamel surfaces with oil-free cleaning paste.
2. Rinse well removing all residual paste from pits and fissures.
3. Prepare questionable enamel in the usual manner.
4. Isolate treatment area and dry thoroughly with oil-free and moisture-free compressed air.
5. Apply Etch-Rite™ 38% phosphoric acid etch gel to the clean, dry treatment site for 20 seconds. Rinse well. Do not disturb this surface.
6. Dry etched surfaces with clean, uncontaminated compressed air. Avoid saliva contamination. If etched surfaces do not appear frosty or chalky white, repeat steps 5 and 6.
7. A drying agent may be applied to etched surfaces at this time. Do not disturb these surfaces. Salivary contamination must be avoided at this critical point. If saliva contamination occurs, wash and dry surface, and etch again with Etch-Rite for 10 seconds. Wash and dry again and proceed.
8. Place a 22-gauge applicator tip securely on Seal-Rite syringe and carefully start flow of Seal-Rite. Use a new applicator tip for each patient.

9. Apply Seal-Rite to the pits and fissures. Flow the material from cusp to cusp, but do not cover the marginal ridges.
 10. Light cure.
- Note:** Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>
- CAUTION**
 Uncaured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves. Do not exceed manufacturer's recommended curing time for the light you are using.
- STORAGE AND HANDLING**
- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
 - Shelf life of unopened product: 2 years from date of manufacture.
 - Re-cap immediately after use.
- REPORTING OF SERIOUS INCIDENTS**
 Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:
 Pulpdent Corporation
 80 Oakland Street, Watertown, MA 02472 USA
 Tel: 617-926-6666 / Fax: 617-926-6262
 email: pulpdent@pulpdent.com
Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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